




# KaysenBetten

Care cots for children

... the original!



Assembly Instructions  
for your new **Carsten I** - care cot

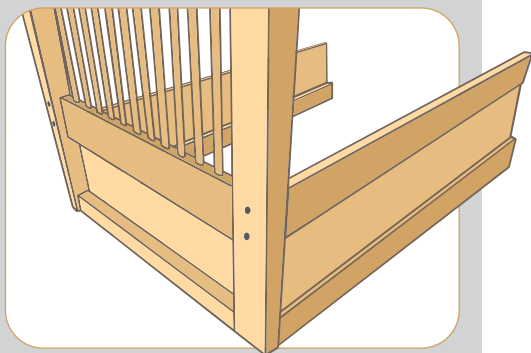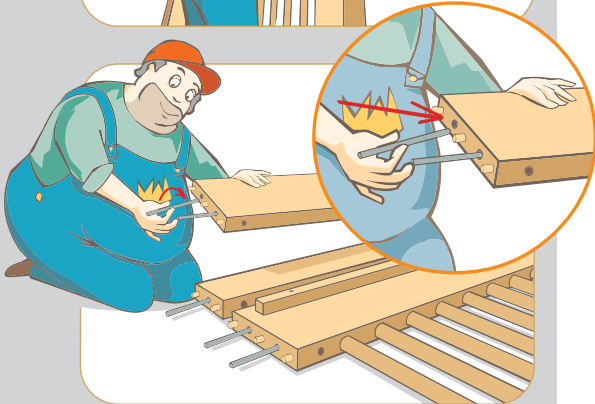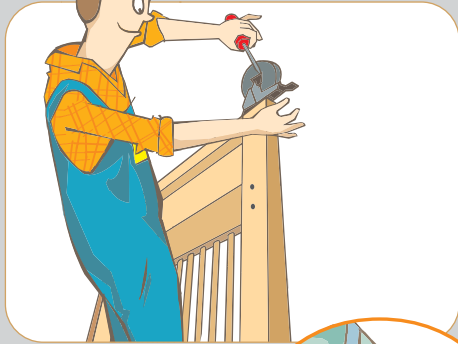

Hello!

Nice to meet you!



**These are our mechanics  
Robert and Stefan.  
They will help you  
to assemble your KayserBed!**

## Assembly Instructions for your new **Carsten I** - care cot

If your bed is equipped with a set of castors then please screw them first into the holes drilled under the side frames (drawing left above).

Please lay the architraves ④ + ⑤ and the back part ③ on the ground in order to push in the transverse nuts ⑨. Now screw in the threaded bars ⑩.



Set up one side frame ①. At first, install the first architrave ⑤ by putting it with the threaded bars through the lower drilled holes of the side frame. The wooden pegs must fit properly into the drillings. Bolt the architrave with a brass cap ⑪. Now, the construction stands steady already!

Install the second lower architrave ⑤.

Now the apertures ⑥ have to be put into the grooves of the installed architraves. At the front side insert the upper architrave ④ and at the back side insert the back part ③ with the threaded bars into the upper drilled holes of the side frame. The batten need to be inside at the bottom. Screw the architrave and the back part with two brass caps each.

Now the second side frame ① has to be pushed onto the threaded bars of the connecting architraves and the back part.

Secure all threaded bars with tightly screwed brass caps.

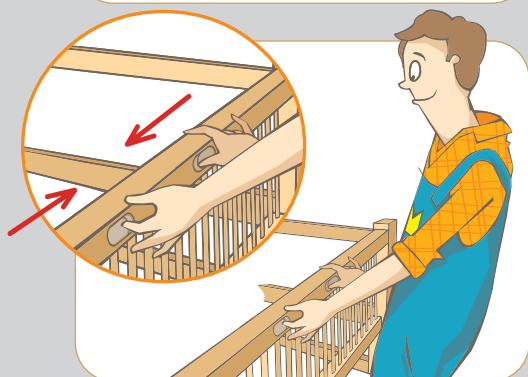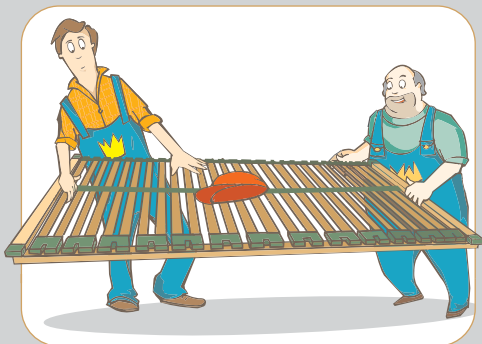

The bed is now ready so that the sleeping platform (13) can be inserted. At first, take the control units out of the packaging. Then you need a second person to help with the sleeping platform. To avoid pinching your fingers, hold it from the inside. The sleeping platform needs to be led carefully between the side frames and is laid down onto the wooden strips of the architrave and the back part.

(If your cot is electronically adjustable, then please make sure that the cables are underneath in order to avoid them being trapped. Install the switch box (15) with the screws provided (17) on the left or right exterior of a side frame (out of reach of the patient). For further information concerning the sleeping platform, please read the enclosed operating instructions.)

Now the lowerable cot side (2) should be inserted from above. By pulling the bolting device inward, the cot side can be lowered to different heights.

Now your new Carsten I -cot is ready to use.

## Parts list and explosion views of your new Carsten I - care cot



Pos.	Qty.	Designation
<b>Wooden surround</b>		
1	2	side frame
2	1	lowerable cot side
3	1	back part
4	1	connecting architrave 150mm
5	2	connecting architrave 90mm
6	2	aperture
7	4	castors
<b>Fittings</b>		
8	1	plug-in peg
9	14	transverse nuts
10	14	threaded bars M8x110mm
11	14	brass caps
12	1	allen key
<b>Sleeping platform</b>		
13	1	sleeping platform with slatted frame
14	1	manual control switch*
15	1	security switch box*
16	1	mounting for power cable*
17	2	screws for switch box*
* (only with electronic sleeping platform)		
<b>Optional</b>		
18	1	lifter

## KayserBetten

Rieper Straße 12  
29683 Dorfmark

Telefon: +49 5163 / 6595

Telefax: +49 5163 / 2076

Internet: [www.kayserbetten.de](http://www.kayserbetten.de)

E-mail: [info@kayserbetten.de](mailto:info@kayserbetten.de)

