





KaysenBetten

Care cots for children

... the original!



Assembly Instructions
for your new **Carsten II** - care cot

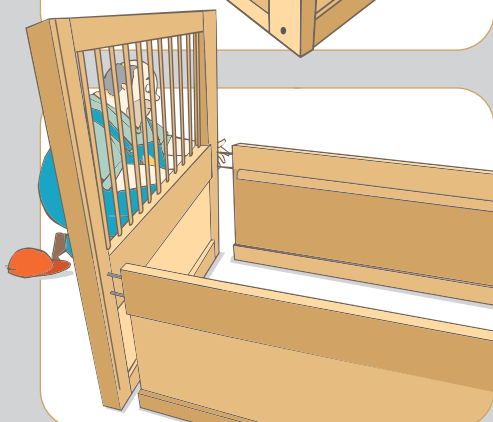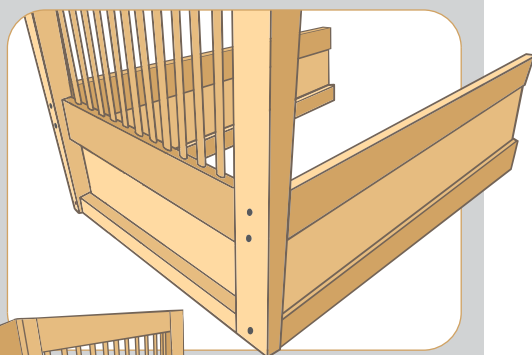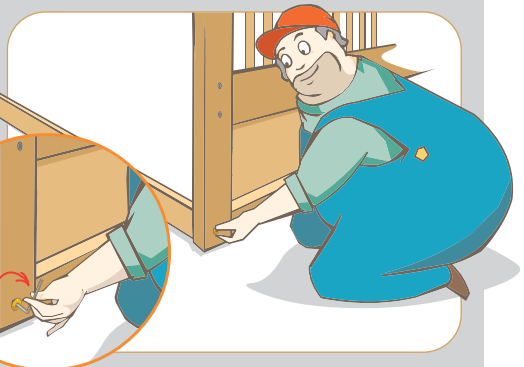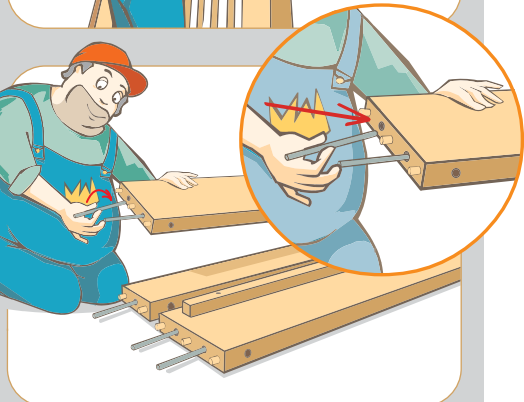

Hello!

Nice to meet you!



**These are our mechanics
Robert and Stefan.
They will help you
to assemble your KayserBed!**

Assembly Instructions for your new **Carsten II** - care cot

If your bed is equipped with a set of castors then please screw them first into the holes drilled under the side frames (drawing left above).

Please lay the architraves ③ + ④ on the ground in order to push in the transverse nuts ⑧. Now screw in the threaded bars ⑨.



Set up one side frame ①. At first, install the first connecting architrave ④ by putting it with the threaded pins through the lower drilled holes of the side frame. The wooden pegs must fit properly into the drillings. Bolt the architrave with a brass cap ⑩. Now, the construction stands steady already!

Install the second lower architrave ④.

Now the apertures ⑤ have to be put into the grooves of the installed architraves, then insert the upper architraves ③ with the threaded bars into the upper drilled holes of the side frame. The batten need to be inside at the bottom. Screw the architraves with two brass caps each.

Now the second side frame ① has to be pushed onto the threaded bars of the connecting architraves.

Secure all threaded bars with tightly screwed brass caps.

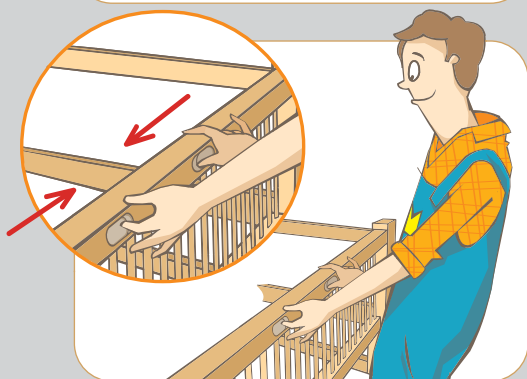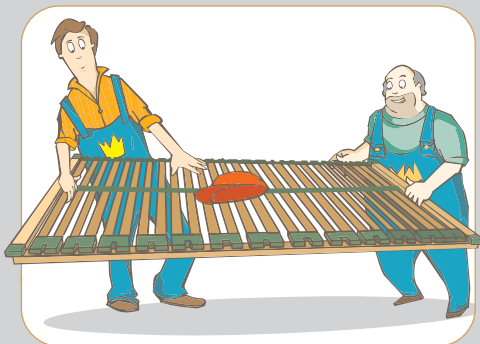

The bed is now ready so that the sleeping platform (12) can be inserted. At first, take the control units out of the packaging. Then you need a second person to help with the sleeping platform. To avoid pinching your fingers, hold it from the inside. The sleeping platform needs to be led carefully between the side frames and is laid down onto the wooden strips of the architraves.

(If your cot is electronically adjustable, then please make sure that the cables are underneath in order to avoid them being trapped. Install the switch box (14) with the screws provided (16) at the left or right exterior of a side frame (out of reach of the patient). For further information concerning the sleeping platform, please read the enclosed operating instructions.)

Now the 2 lowerable cot sides (2) should be inserted from above. By pulling the bolting device inward, the cot sides can be lowered to different heights.

Now your new Carsten II cot is ready to use.

Parts list and explosion views of your new **Carsten II** - care cot



Pos.	Qty.	Designation
Wooden surround		
1	2	side frame
2	2	lowerable cot sides
3	2	connecting architrave 150mm
4	2	connecting architrave 90mm
5	2	apertures
6	4	castors
Fittings		
7	2	plug-in pegs
8	12	transverse nuts
9	12	threaded bars M8x110mm
10	12	brass caps
11	1	allen key
Sleeping platform		
12	1	sleeping platform with slatted frame
13	1	manual control switch*
14	1	security switch box*
15	1	mounting for power cable*
16	2	screws for switch box*
*(only with electronic sleeping platform)		
Optional		
17	1	lifter

KayserBetten

Rieper Straße 12
29683 Dorfmark

Telefon: +49 5163 / 6595

Telefax: +49 5163 / 2076

Internet: www.kayserbetten.de

E-mail: info@kayserbetten.de

